

ENGLISH

DIRECTION FOR CONDOM USE BEFORE SEX:

Use a new condom every time you have sex - and before intimate contact (before foreplay, before penis gets anywhere near any body opening) to avoid exposure to any body fluid that can carry infection. Lesions, pre-ejaculate secretions, semen, vaginal secretions, saliva, urine and faeces can all transmit disease organisms.

Open the pack carefully so that the condom does not tear. Avoid contact with finger nails, jewellery etc for safety.

The condom is best worn on a fully erect, hardened penis. Leave an empty space at the end of the condom to collect semen. Squeeze the tip of the condom and slowly insert the rim onto your hardened penis.

Continue squeezing the tip while slowly unrolling the condom. If the condom does not unroll, it is on wrong. Throw it away. Start over with a new one. Make sure that your condom fits snugly on your penis before any contact occurs between the penis and your partner's body, to assist in the prevention of sexually transmitted diseases and pregnancy.

AFTER SEX:

After you have ejaculated, hold the rim of the condom and pull your penis out before it becomes flaccid. Turn and move completely away from the vagina before you let go of the condom. Do Not Reuse Condoms. Dispose the condom hygienically by wrapping it in tissue and dispose in a waste bin or bury it. If the condom breaks and semen spills or leaks out, do not panic, but wash semen away with soap and water.

Never let a latex condom touch oil in any form - no petroleum jelly, no baby oil, no mineral oil, no vegetable oil, no massage oil- oil rots rubber, no body lotions, no butter no margarine, no mustard oil, etc. (i.e., no oil based lubricants) as these can damage the condom. Avoid contact with talcum/baby powder because it may contain oil.

In case of use of any topical medications (on the penis or the vagina) please consult a doctor or a pharmacist before use.

For additional lubrication you may use personal lubricants, designed for use with condoms or other water - based lubricants.

Wash hands - as well as penis, vagina and surrounding areas - before and after sex and take special care to wash off lubricant after use of condom. This cuts the chance of infection.

Avoid contact with eyes

This product should not be used if you or your partner has any inflammation or broken skin.

Keep unused condoms in their packs in a cool, dry place away from direct sunlight (not in a wallet)

If a new condom feels sticky or stiff or looks damaged in any way, throw it out - use a fresh one.

(*Methaemoglobinaemia risk stated in lay terms)

For single use only

IMPORTANT

If used properly, latex condoms will help to reduce the risk of transmission of HIV infection

(AIDS) and many other sexually transmitted disease including chlamydia, genital herpes, genital warts, gonorrhea, hepatitis B and syphilis.

WARNINGS:

Caution: This product contains rubber latex which may cause allergic reactions. Allnight and Absolute Xtasy variants have a special lubricant which contains 4% Benzocaine in Poly Ethylene Glycol (PEG). Some users are sensitive or allergic to lubricants. If you are or your partner have had any reaction to latex rubber or lubricants, do not use this product. If either partner develops a rash or irritation, stop use and consult a doctor.

If either partner experiences respiratory difficulty or develops blue lips following use of this product, seek medical advice immediately and stop using the product.